

WAFFLE® Overlay QUICK CHECK



Indications for Use

Treatment: Use for healing therapy up thru Stage IV and unstageable pressure ulcers. Deep tissue injury protection.

Prevention: Pressure ulcer prevention for all levels of risk.


Comfort: Use on standard hospital bed for patient comfort.

Recommended for use: hospital beds, ER carts, post-surgery, turning, transferring and positioning patients, and at home.
Single patient use.

Dogbone shaped handwells facilitate easy patient handling.



For patient and caregiver safety, let the WAFFLE® Mattress work as a mobilizing, turning and transferring device!

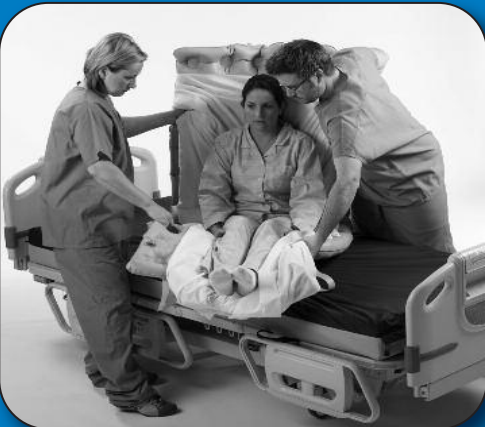

MOBILIZING



TRANSFERRING



TURNING

WAFFLE® Overlay QUICK CHECK

Unfold the overlay with the valve on the top and positioned at the foot of the bed.

1.



Secure the straps under the corners of the bed.

2.




Open clear valve and insert the pump to inflate. **Pull Out and Push In for One Complete Stroke.**

3.




Close the valve tightly.

4.

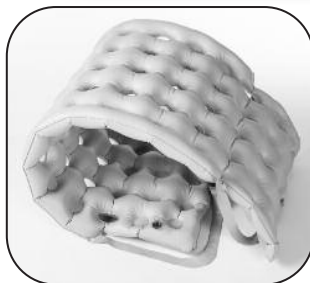

Too Full

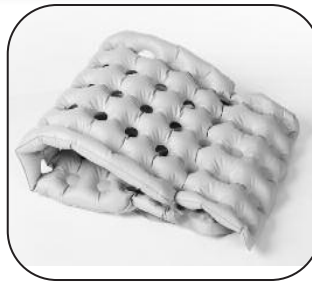

Correct

The mattress overlay should only look 60% full. When it is properly inflated, the overlay should easily fold in half. If the overlay has to be forced to fold, the product has too much air.

Hand Check & Inflation Adjustments



The key to safe patient support is creating the perfect balance between pliability and stiffness of the mattress. If a mattress is too stiff, the advantages of the surface area are defeated. One inch of air is adequate support for the patient.

If you can slide your hand in and out easily under the overlay, pressure is being relieved. Although you may feel the patient, your hand has displaced the air that has been supporting the patient. The purpose of the hand check is to adjust the air in the product, if necessary. EHOB recommends hand checks at every shift and then adding or removing air, according to the patient's position and the body type.